



### OLYMPIC CFM 6000 Version 2.0 Highlights

#### On-Screen Displays and Controls

- Selectable CFM Insight™ display **NEW**
- CFM, Impedance, and EEG Views
- Touch-screen controls

#### Review EEG

- Live EEG **NEW**
- EEG position indicator
- Expanded EEG views: time (mm/sec) and amplitude (µV) **NEW**

#### Marker Events

- Quickly mark significant events
- Improved marker functionality, including marker list review **NEW**
- More standard markers or enter a custom marker **NEW**

#### Print Data

- Built-in thermal printer
- Faster print speed
- More print options **NEW**

#### Manage Files

- Archive, restore, or delete files
- Save more data to a CD **NEW**
- Span long files over several CDs **NEW**

#### Data Entry

- Add patient information to file
- Easier data entry: drop-down and scroll select
- Auto-capitalization choices for text-entry fields **NEW**

#### Built-In Safeguards

- Self calibration check
- Lead-off and high impedance notification
- Option to resume previous patient trace after power interrupt **NEW**

#### Training/Guidance

- Expanded online help with pictorial instruction for electrode placement **NEW**

#### Language Selection

- Multiple language selection
- Easier to change languages **NEW**

### Device Update Instructions

**⚠ Caution:** Choosing install or upgrading from v. 1.8 will overwrite the device's operating system, including all patient files. If performing an install or upgrading, archive all patient files that you want to retain **prior to installing the software.**

#### To update from version 1.8:

1. Turn on your CFM 6000.
2. Touch **Tools**, touch **Update**.
3. Insert the CD in the tray, touch **OK**.
4. The system will restart and load the new software. When the update is complete, remove the CD from the tray and return it to this case.

#### To update from version 2.0:

1. Turn on your CFM 6000.
2. Touch **Tools**, then **System**, then **Software Update**.
3. Touch **Open Tray**. Place the CD in the tray, touch **Close Tray**.
4. Select whether you are upgrading the software or installing a new language, then touch **Start Update**.

### Viewer Installation Instructions

*System Requirements: Windows 2000, XP Home, or XP Pro.*

1. Place the CD in your computer.
2. If the CD doesn't automatically start, click **Start**, click **Run**, then browse to the CD location. Select setup.exe, then click **OK**.
3. In the Run window, click **OK**.

**For information or ordering**  
**Call 1-800-426-0353**  
(toll-free in US/Canada)

To find your local Olympic CFM distributor, outside the United States and Canada, call (01) 1-206-767-3500.


 Non-English users: Refer to your Olympic CFM 6000 Operator's Manual.




OLYMPIC CFM 6000 Version 2.0

# OLYMPIC CFM 6000

## Version 2.0



Software Update/Install and CFM Viewer CDS



### aEEG Infant Brain Monitor

### Software Update/Install CD

### and CFM Viewer CD

Now...with CFM Insight™