

OLYMPIC COOL-CAP® Setup

34.9 Rectal °C

OK



1. Temperature Sensor Module

34.9°C



2. Rectal Temperature Sensor



3. Scalp Temperature Sensor



4. Skin Temperature Sensor and Radiant Warmer Servo Sensor



5. Optional: Radiant Warmth Temperature Sensor



6. Optional: Temperature Sensor Name: Other



*TSM photo w/
rectal plugged in*



A. Plug the rectal sensor into the temperature sensor module.

B. Apply lube to the rectal sensor and slide it 5 cm into the patient's rectum. The sensor is marked at 6 cm.

C. To keep the sensor in place, tape the rectal sensor wire to the patient's leg.

[Next](#)

Setup Page:

1

2

3

4

5

6

[Exit](#)[Patient Data](#)[Clinical Data](#)[Sensors](#)[Connections](#)[Water Flow](#)[Caps](#)[?](#)